



Foremount Enterprise Co., Ltd.

www.foremount.com

Silicone Mask

Instruction for use



Caution:

- The orifice of the mask is designed connecting with 22 mm OD or 15mm ID device.
- Make sure connection is secure.
- Please replace immediately if silicon mask is rupture.
- The cumulative use of the device on patient should not exceed 24H
- If patient have an allergy and other adverse reactions, please remove the device.
- Storage temperature: 15°C ~ 25°C

INSTRUCTIONS FOR USE:

1. Read the manual carefully before use.
2. Remove mask from the package before use.
3. Connect mask to breathing circuit or other device.
4. Check the mask and put the mask on the nose and mask in the right position.
5. Fix the harness with the mask without coming off.
6. Put the harness behind the brain, and then make sure the harness is Fixable and comfortable for the patients.



CLEANING AND DISINFECTION INSTRUCTIONS:

CLEANING:

1. Rinse in lukewarm (< 43 °C) clean water before exposure to detergent.
2. Wash the mask with clean warm water and neutral detergent.
3. Remove all detergent and rinse the mask in clean water.
4. Dry thoroughly and package the mask.

DISINFECTION Methods:

- (1) Autoclaving: Autoclave up to 121°C around 30 minutes. Maximum number of Autoclave Cycles for each mask: 25 times.



Foremount Enterprise Co., Ltd.

www.foremount.com

Imported and distributed by:



Oscar Boscarol srl

E. Ferrari Street, 29 – 39100 Bozen

Phone. 0471932893 – Fax. 0257760140


Email: info@boscarol.it

Produced by:



Foremount Enterprise Co., Ltd

No.17, Aly. 15, Ln. 5, Shenan St., Shengang Dist., Taichung City 42944, Taiwan
(R.O.C.)

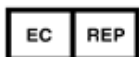

Obelis S.A.

Bd. Général Wahis 53 1030 Bruxelles, Belgium.



Version 5, issue date: 2020/04/28