

CE 2460

Disposable Aircushion Mask

REF

LOT



Disposable for single use. Not intended for reprocessing.
It may cause cross-infection if reused.

Side Effects:

- Pressure sore.



Caution:

- This device is intended for use only by adequately trained medical personnel.
- After using, the method of waste disposal according to the hospital infection control policy.
- If patient have an allergy and other adverse reactions, please remove the device.
- Do not process or reprocess.
- The cumulative use of the device on patient should not exceed 24H.



- Keep away from sunlight.
- Keep dry.



- Do not use if package is open, wet or damaged.
- Storage temperature: 15°C ~ 25°C.



Instruction For Use:

- Read the manual carefully before use.
- Open the bag before use.
- Check the mask and assure the cushion is filled with air suitably.
- Fix the harness with the mask without coming off.
- Put the mask on the nose and mask in the right position.
- Put the harness behind the brain, and then make sure the harness is fixable and comfortable for the patients.
- Make sure there is no leakage when usage.
- Connect the circuit with mask and gas source device.

Imported and distributed by:



Oscar Boscarol srl

E. Ferrari Street, 29 – 39100 Bozen
Phone. 0471932893 – Fax. 0257760140
Email: info@boscarol.it

Produced by:



Foremount Enterprise Co., Ltd

No.17, Aly. 15, Ln. 5, Shenan St., Shengang Dist., Taichung City 42944,
Taiwan (R.O.C.)



Obelis S.A.

Bd. Général Wahis 53 1030 Bruxelles, Belgium.