



## DISPOGEL

### HOT/COLD GEL 14x24 CM

- Reusable gelatin hot/cold Dispo GEL
- It is a hot/cold reusable accumulator from outside source.
- Completely non-toxic, suitable for medical and domestic use: easily modeled, does not harden becoming ice and it is reusable.
- Cold therapy: keep in freezer for at least 2 hours. Take out of the freezer and put in a cloth and apply on the affected part.
- Hot therapy: place the bag in the microwave for at most 1 minute setting it to 650 Watt or dip the bag in boiling water for at most 5 minutes, remove and dry; in both cases insert the bag in a cloth and apply on the affected part.

For further information and/or technical data related to the product, please refer to the Oscar Boscarol Company.