Medizinischer Gebrauch	Medical use	Uso medico	Usage medicinal
<b>DRUCKREGLER</b>	<b>PRESSURE REGULATOR</b>	<b>RIDUTTORE DI OSSIGENO</b>	<b>DÉTENDEUR DE PRESSION</b>
Gebrauchs- und Wartungshand- buch	User and Service Manual	Libretto di istruzioni d'uso	Manuel d'Usage et Maintenance
Deutsch	English	Italiano	Français

CE  
0051

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## ENGLISH

- ⚠ *Escaping oxygen increases the risk of fire. Never smoke near equipment which carries oxygen and avoid using open fires on glowing objects. Check tight fitting and firm seat of the connector when mounting accessories.*
- ⚠ *Do not expose the oxygen cylinder to intense heat. Do not position close to radiators and protect from high solar radiation.*
- ⚠ *Compressed oxygen is very dense. Opening of the cylinder valve quickly may produce a pressure surge which will cause an explosive flame by igniting dirt particles or residues of disinfectants. Open the cylinder valve of the oxygen cylinder slowly.*

### Connecting the regulator to the cylinder:

Proceed following the steps:

1. Make sure that the seal of the valve of the cylinder has been removed and that is not dirty, has no signs of oil or lubricant; eliminate it in case there is some.
2. Slowly thread the connection of the regulator of the cylinder tightening it by hand without using any tools.
3. Connect the accessories by their ends.

### Use:

1. Once you have connected the regulator to the cylinder and accessories, **SLOWLY OPEN THE CYLINDER'S VALVE**.
2. Make sure that the pressure increases progressively in the pressure gauge and stabilize it when you open the valve all the way.
3. If you notice any escapes, quickly close the valve of the bottle and specially check the joint in the connection of the bottle, in case it has one.
4. In the case of the regulator with flowmeter (see models), turn the remote counterclockwise to increase the value of the flow given.
5. **TO REMOVE THE REGULATOR FROM THE CYLINDER, CLOSE THE VALVE OF THE CYLINDER BEFOREHAND.**

### How to maintain and clean it:

- The equipment and its accessories must never come in contact with any grease or lubricant, or any substance that could contain them.
- The equipment must be used by specialized and medical personnel. The equipment's maintenance must be handled by qualified personnel and authorized by the manufacturer.
- The frequency of the inspections will depend on the time of use of the equipment, but must never exceed five years.
- In case of any anomaly of the equipment, please contact the manufacturer for its repair.
- To clean the equipment - if this is necessary - you must remove the particles and dirt that may have been accumulated in the connection area of the bottle. Never use liquids that could enter into the equipment.
- Leave the equipment in a dry and clean place and where it can be protected from dents. Always store the equipment within the following temperature range: -20 to 60°C.

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